



Rehab car seat KIDSFLEX

...diverse, as the needs of your child

The KIDSFLEX child car seat, specially designed for rehab, has adjustable and individually positionable trunk supports that control the child's lateral movement. The following accessories are optionally available to ensure comfortable and safe transport in the car:

- angular Adjustment
- swivel adapter
- leg supports
- ISOFIX adapter
- security tray
- abduction block
- different head supports
- foot rest
- trunk supports
- shoulder supports
- security pad for the buckle

With these options, individual needs of the individual child can be considered, especially since no accessory excludes another. As many modern vehicles are equipped with ISOFIX, we offer a suitable adapter system to fix the seat optimally in the car. An important innovation is the 5-point harness, which can now be tightened with just one movement, making everyday life much easier.



quick-release fastener for the 5-point harness



steppless angular adjustment



head rest Kick-Back with patented bending points



foot rest angle- and height-adjustable

... is flexible, comfortable and safe

5-point harness with quick-release fastener

Back angle adjustment

Patented "Kick-Back" headrest

Many innovative accessories

HERNIK GmbH Auf der Gröb 2d 83064 Raubling GERMANY

** +49 (0)8035-9580448 ** +49 (0)8035-9580446

⊠ info@hernik.de

www.hemik.de

Rehab car seat KIDSFLEX

... a car seat for children with special needs

The KIDSFLEX rehab car seat was designed as a restraint system for physically and/or mentally disabled children in cars. It is suitable for installation in most commercially available vehicles. The KIDSFLEX is built for use in the back seat of a passenger car and requires a functioning 3-point safety belt. The seat can be fitted behind the front passenger or - by converting the belt quide - behind the driver's seat.

In addition to the 3-point harness of your car, the KIDSFLEX is equipped with its own 5-point chest-shoulder harness, which provides additional support for your child.




The adjustment of the seat depth, as well as the variable belt quide in the backrest, offer sufficient possibilities to adapt the car seat accordingly as the child grows. You can choose from a wide range of accessories to suit your child's individual needs.

Even if the three different sizes of the KIDSFLEX do not match your child's stature, we offer a flexible solution for this: we change the corresponding size individually!

A very popular feature is the innovative swivel adapter, which makes the transfer of the increasingly heavy child much easier. The pivot point has been moved from the centre to the rear and outwards in order to ensure that the chair moves away from the back upholstery of the car and towards the door cut-out at the same time. Great importance was attached to a compact design: the overall height of the seat remains unchanged!

The part of the body that travels the greatest distance in a crash is the head. For this reason, the "Kick-Back" head restraint was developed, which optimally absorbs the kinetic energy of the head through patented predetermined bending points and thus prevents dangerous injuries.



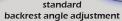



seat depth adjustment

- KIDSFLEX in size 2, black/red with soft head support, trunk supports, shoulder supports, leg supports, abduction bloc angular adjustment and swivel adapter.
- KIDSFLEX in size 2, black/blue with soft head support, trunk supports and swivel adapter
- Infinitely lockable back angle adjustment (standard)
- Continuous seat depth adjustment over 6 cm (standard)